TMS300RDR-V01-EI-02/2010

TMS 300 RDR

MOBILE IMAGING SYSTEM

UNI EN ISO 9001:2008

UNI EN ISO 13485:2004

TECHNIX SPA

24050 Grassobbio (Bg) - Italy
via E. Fermi, 45
Tel. +39.035.38.46.611
Fax +39.035.33.56.75
technical@technix.it
www.technix.it

TECHNIX

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TECHNIX

The TMS 300 RDR is a mobile x-ray unit which suits a wide range of clinical applications.

Its compactness and manoeuvrability makes the unit movement smooth and precise through doors, wards and elevators even in the hospital's small rooms.

The ergonomic design allows the operator to have an excellent visibility during the transportation.

The monobloc's arm, which is perfectly balanced, allows a free movement and easy positioning even in the most critical conditions.

The basement shape allows the easy positioning under the patient bed. TMS 300 RDR has a digital image acquisition system in radiograph-

ic mode, that uses a solid detector of big dimensions, commonly named flat panel detector (FPD).

The acquired images are saved in the memory of the system, displayed on a high resolution monitor and eventually sent to a diagnostic work station or to a general archive through DICOM connection.

The unit provides the specialization of the parameters of acquisition, process and image display according to both the selected anatomic part and the patient size. This allows a correct and immediate acquisition and display of the images, avoiding subsequent improvement operations. Also sophisticated post-processing operations are available in order to improve images and facilitate their management, such as: dedicated LUT for every examination, spatial filters, harmonization, V/H image inversion, 90° image rotation, electronic

rectangular and circular collimators, grey scale inversion, multi-image display, images increase with scale factor variable from 1 to 3, brightness and contrast variation, grid display, distances calculation, angles calculations, text overwriting on the image.

TMS 300 RDR is fitted with three systems for images printing and archiving:

1. Direct print through protocol 3M-M959: it allows to print images on laser (single or in overview) located on the monitor. All principal laser printers are compatible with the system.
2. Archive on CD with DICOM format (optional): this function is immediately available both from the patient list and from the examination.
3. DICOM Network connection (optional): it allows to archive (store) on workstation and print. The "store" function archives the single

image, displayed on the monitor or even all the study. The "print" function allows to print the x-ray film, which has been created by the option Film Editor.

The unit is intended to be used mainly to perform diagnostic and radiological examinations of:

- Intensive care
- First aid
- Orthopaedics
- Paediatrics
- Neonatology
- Sports medicine
- Operating theatre